



PRODUCT PERFORMANCE (STRUCTURAL COMPOSITION):

This mask is composed of fibrous material specifically engineered to comply with particulate standard, N95/KN95/FFP2 commonly recommended by health authorities during pandemic and or other emergency situations.
*not suitable for fire or steam.

SCOPE OF APPLICATION:

This mask is suitable for all types of occupational use while covering the user's mouth, nose and jaw. This mask will function to provide a specific physical barrier to prevent particulate matter, etc. from directly penetrating the mask itself.

INSTRUCTION FOR USE:

Unfold the mask with the nose clip facing upwards, then attach the strap to the ear and adjust to fit into the nose clip.

PRECAUTIONS:

Check the packaging before use. Damaged packaging VOIDS mask for use. This mask is for ONE-TIME use only. If you have any discomfort such as breathing difficulty- immediately remove and discontinue use of mask.

DISPOSAL:

Mask should be considered and treated as medical waste to prevent environmental pollution.

STORAGE CONDITIONS:

This product should be stored in a non-corrosive gas, cool, dry, well-ventilated environment, and away from fire and or flammable items.

DIMENSIONS:

6.3in x 4.21in (16cm x 10.7cm)

PRODUCTION DATE, PRODUCTION BATCH NUMBER, VALIDITY DATE:

See packaging

CERTIFICATION:

FDA Approved

MEDICAL DEVICE PRODUCTION FILING CERTIFICATE NUMBER:

To be included on packaging.

**DISPOSABLE
FOR ONE USE ONLY**



USAGE: As shown below

